


# Heck Anterior Modular Hip Component Extractor with Strikeplate

Designed by David Heck, MD

## Strikeplate provides additional help to remove a femoral hip stem

In this process of placing the extractor over the neck and tightening the locking screw, the upper flange surface of the strikeplate can be hit to help engagement. The inferior flange surface of the strikeplate can be hit in a vertical fashion when the femoral component is particularly well engaged. The extractor is equipped with a swivel block for attachment of a slap hammer. The swivel block helps keep the slap hammer in line with the angle of the femoral stem. Includes standard slap hammer, #3925.

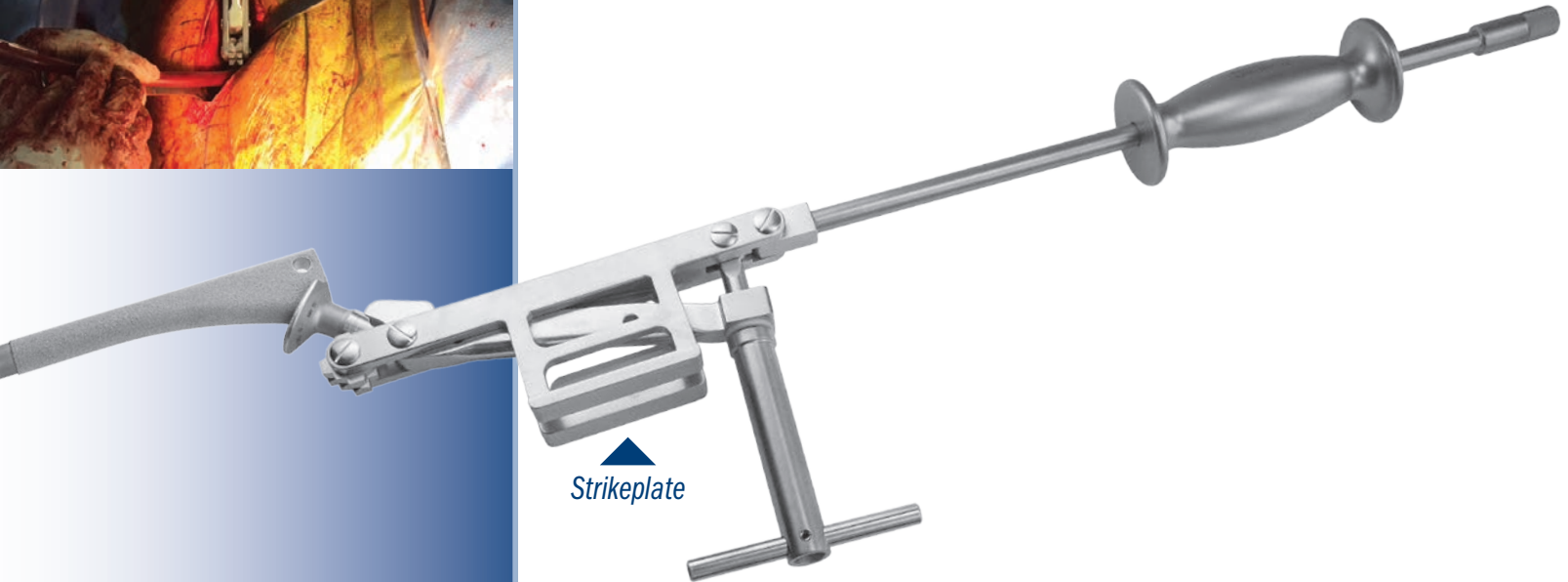

Strikeplate

PRODUCT NO'S:	
3611	[Extractor w/Std. Slap Hammer #3925]
Optional/Individual Parts:	
3611-01	[Extractor Only]
3925	[Standard Slap Hammer] 3/8"-16 Thread Gauge
3935	[Extra Large Slap Hammer] 3/8"-16 Thread Gauge



ISO 9001:2008 • ISO 13485:2003

FREE TRIAL ON MOST INSTRUMENTS

Scan to  
Launch Our  
Website



103 Estus Drive, Savannah, GA 31404  
www.innomed.net info@innomed.net

912.236.0000 Phone  
912.236.7766 Fax

Innomed-Europe Tel. +41 41 740 67 74  
Fax +41 41 740 67 71



1.800.548.2362