

1-800-548-2362

International (912) 236-0000



www.innomed.net

(912) 236-7766 Fax

©2004 Innomed, Inc., All Rights Reserved.

Innomed

103 Estus Drive, Savannah, GA 31404
E-Mail: info@innomed.net



FREE TRIAL ON MOST INSTRUMENTS
CALL FOR MORE INFORMATION

Incision Aligner

The bent ends of the aligner are placed at each end of an incision, which is aligned by pulling outward on each end. The sliding end will lock in place when it is tensioned. Pressing inward slightly on the sliding end will allow the aligner to be collapsed and removed.

Designed by
DMP

Product No: 1330

Designed to align an incision during closing

