

Trans-sulcus Angle Guide

Helps establish the trans-sulcus line

A line is drawn down the deepest part of the trochlear sulcus (Whiteside line) with a marking pen or cautery. The post on the guide is inserted into the hole in the femur made for an intra-medullary alignment guide. The trans-sulcus angle guide is then rotated until the line on the guide lines up with the Whiteside line. A line is then drawn along the bottom of the guide.

Product No: 1160
Dimensions: 2.25" x .75"
Post Depth: 1.5"



FREE TRIAL
ON MOST INSTRUMENTS
CALL FOR MORE INFORMATION



©2009 Innomed, Inc., All Rights Reserved

Phone **800 548 2362**
912 236 0000
Fax 912 236 7766
Internet www.innomed.net
eMail info@innomed.net

Innomed
ORTHOPEDIC INSTRUMENTS
103 Estus Drive Savannah GA 31404

