

## Wubben Guide Rod Pusher


*Used to help hold the guide rod in place during a intramedullary nailing of a long bone*



The surgeon can use the concave end of handle initially to hold the guide rod in place. As the reamer is retracted to the end of the guide rod, the shaft of the Pusher is used by inserting down the center hole of the drill, pushing on the guide and keeping it in the bone.

Designed by  
Robert Wubben, MD

Product No: 5985  
Overall Length: 9"  
Pusher End Diameter: 3.3mm  
Handle End Diameter: 10mm

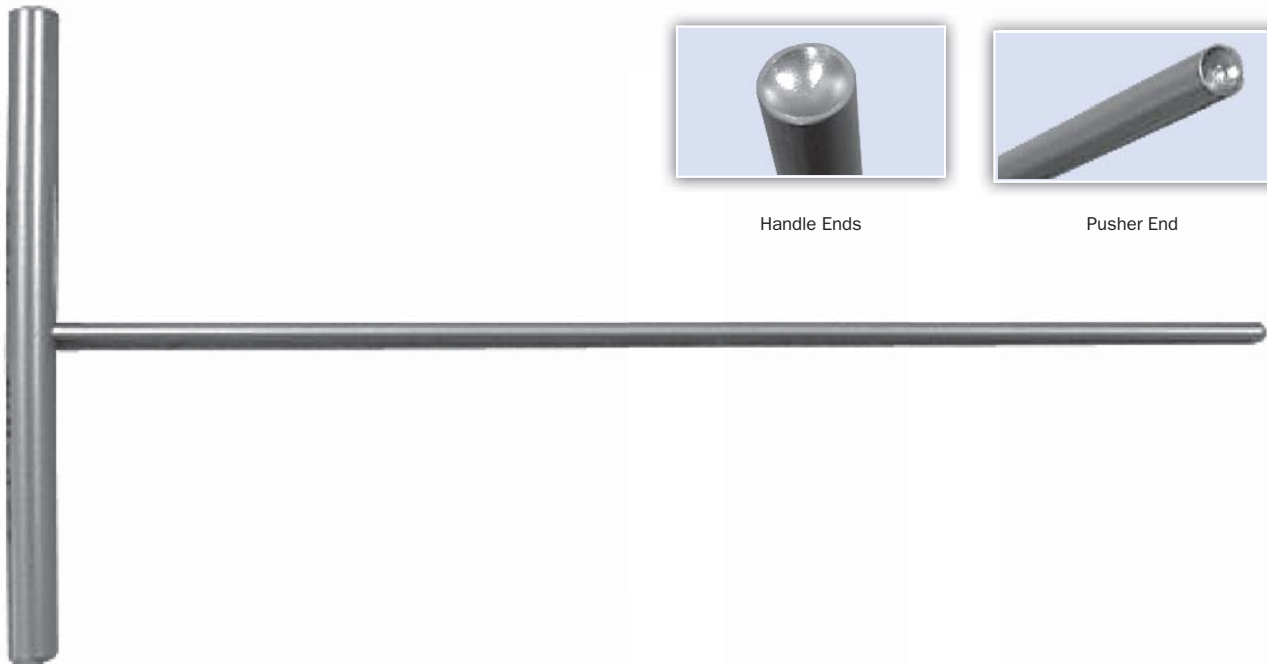

Handle Ends



Pusher End

**FREE TRIAL**  
ON MOST INSTRUMENTS  
CALL FOR MORE INFORMATION



©2008 Innomed, Inc., All Rights Reserved

Phone **800 548 2362**  
912 236 0000  
Fax 912 236 7766  
Internet [www.innomed.net](http://www.innomed.net)  
eMail [info@innomed.net](mailto:info@innomed.net)

**Innomed**  
ORTHOPEDIC INSTRUMENTS  
103 Estus Drive Savannah GA 31404

