

# 2620-C Linear Position Assembly

Disassembly instructions for cleaning or sterilization purposes

1) Remove Adjustable Handle from Boot Lock Sub Assembly

Figure 1



2) Remove Boot Lock Sub Assembly from Radial Body

Figure 2




3) Remove Prong Washer from Boot Lock Sub Assembly

Figure 3



4) Remove Adjustable Handle from Radial Body. Operator may use a screwdriver to remove the Adjustable Handle, or may unscrew the handle while applying down pressure to the spring loaded handle of the Linear Position Sub Assembly.

Figure 4

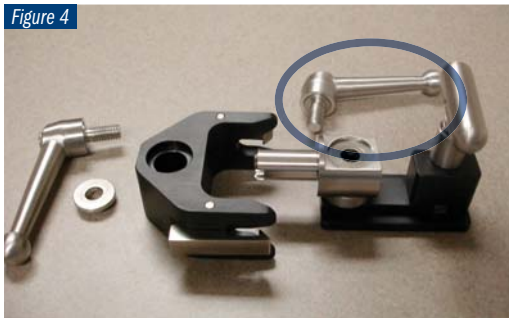


5) Remove Radial Body from Linear Position Sub Assembly

Figure 5



6) Remove Prong Washer from Radial Body

Figure 6

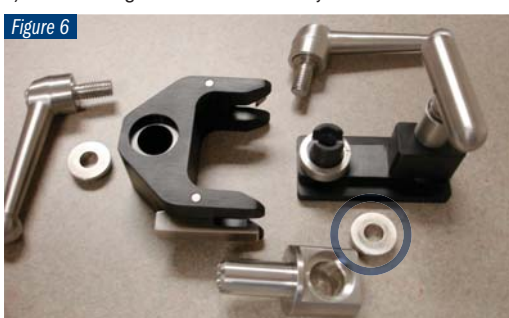

7) Remove Stainless Steel Spacer from Linear Position Sub Assembly

Figure 7



8) Reassemble Linear Position Assembly in reverse order following required cleaning or sterilizing

Figure 8



ISO 9001:2008 • ISO 13485:2003

FREE TRIAL ON MOST INSTRUMENTS



103 Estus Drive, Savannah, GA 31404  
www.innomed.net info@innomed.net

912.236.0000 Phone  
912.236.7766 Fax

Scan to  
Launch Our  
Website



**INNOMED**

Innomed-Europe Tel. +41 41 740 67 74  
Fax +41 41 740 67 71

1.800.548.2362