



INNOMED, INC.

103 Estus Drive
Savannah, GA 31404

Cleaning and Disinfecting Recommendations:

Instruments, pads, and straps may be cleaned with a neutral, liquid enzymatic detergent, such as Endozime, Luminox, ZymeX, etc.

Pre-Cleaning

- Remove gross soil with wipes
- Soak in cleaning solution
- Use brush or sponge to remove soil
- Rinse in running water
- Inspect, place on absorbent paper

Manual Cleaning and Disinfection

- Soak in Ultrasonic bath
- Clean with brush
- Rinse in running water
- Soak in disinfectant solution
- Clean with brush
- Rinse with running water
- Inspect and dry