

## STERILIZATION PROCEDURE

# AccuAngle Indicator

Product No: 1325

Made of aluminum.

*Please use the following guidelines when sterilizing this product:*

| <b>Sterilizer Type:</b>    | <b>Steam Sterilizer, without vacuum</b> |
|----------------------------|---|
| Minimum Temperature:       | 132 degrees C / 270 degrees F           |
| <b>Minimum Cycle Time:</b> | <b>6 minutes</b>                        |
| Minimum Dry Time:          | 20 minutes                              |

Revised 1/1/12

FREE TRIAL ON MOST INSTRUMENTS

© 2012 Innomed, Inc.

**INNOMED** 



103 Estus Drive, Savannah, GA 31404  
www.innomed.net info@innomed.net

912.236.0000 Phone  
912.236.7766 Fax

**1.800.548.2362**