

STERILIZATION PROCEDURE

AccuAngle Indicator

Product No: 1325

Made of aluminum.

Please use the following guidelines when sterilizing this product:

Sterilizer Type:	Steam Sterilizer, without vacuum
Minimum Temperature:	132 degrees C / 270 degrees F
Minimum Cycle Time:	4 minutes
Minimum Dry Time:	30 minutes

Revised 1/10/17

FREE TRIAL ON MOST INSTRUMENTS

© 2013 Innomed, Inc.

INNOMED 



103 Estus Drive, Savannah, GA 31404
www.innomed.net info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

1.800.548.2362