



**INNOMED, INC.**

103 Estus Drive  
Savannah, GA 31404

## **Sterilization Parameters**

**Part No. 2770-01**

### **Adjustable Knee and Tibial Positioner**

The Adjustable Knee and Tibial Positioner is made of propylux, polypropylene, and nylon. Please use the following parameters when reprocessing this device.

Sterilizer Type:	Pre-vacuum Steam Sterilizer
Minimum Temperature:	132 °C / 270 °F
Minimum Cycle Time:	4 minutes
Minimum Dry Time:	30 minutes

#### Silicone Pad

Can be pre-vacuum, steam sterilized separately. May also be cleaned with a liquid enzymatic detergent such as Enzol.

#### Velcro Straps

Can be pre-vacuum steam sterilized separately. It is recommended that used straps are replced.