

STERILIZATION PROCEDURE

Dozier Radiolucent Bennett Hip Fracture Retractor

Product No: 6870

Made of Delrin.

Please use the following guidelines when sterilizing this product:

Sterilizer Type:	Pre-vacuum Steam Sterilizer
Minimum Temperature:	132 degrees C / 270 degrees F
Minimum Cycle Time:	6 minutes
Minimum Dry Time:	0 minutes

Can accept a dry time according to your sterilization equipment's manufacturing instructions for Delrin products.

Can be pre-vacuum steam sterilized **wrapped or unwrapped.**

Revised 1/1/12

FREE TRIAL ON MOST INSTRUMENTS

© 2012 Innomed, Inc.

INNOMED 



103 Estus Drive, Savannah, GA 31404
www.innomed.net info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

1.800.548.2362