

STERILIZATION PROCEDURE

Extremity Positioning Tube, 4" & 6"

Product No: 2740-01 & 2740-02

Made of aluminum.

Please use the following guidelines when sterilizing this product:

Sterilizer Type:	Pre-vacuum Steam Sterilizer
Minimum Temperature:	132 degrees C / 270 degrees F
Minimum Cycle Time:	6 minutes
Minimum Dry Time:	20 minutes

Can be pre-vacuum steam sterilized **wrapped or unwrapped**, assembled with straps and silicone pad in place, or unassembled with straps and silicone pad not attached to the tube.

The Velcro straps may be cleaned with a liquid enzymatic detergent such as Cabco Zyme.

Revised 1/1/12

FREE TRIAL ON MOST INSTRUMENTS

© 2012 Innomed, Inc.

INNOMED 



103 Estus Drive, Savannah, GA 31404
www.innomed.net info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

1.800.548.2362