

STERILIZATION PROCEDURE

Fromm Femur & Tibia Triangle

Product No: 2760

Made of aluminum.

Please use the following guidelines when sterilizing this product:

Sterilizer Type:	Pre-vacuum Steam Sterilizer
Minimum Temperature:	132 degrees C / 270 degrees F
Minimum Cycle Time:	4 minutes
Minimum Dry Time:	30 minutes

Can be pre-vacuum steam sterilized **wrapped or unwrapped**, assembled with straps and silicone pad in place, or unassembled with straps and silicone pad not attached to the triangle.

The Assistant Free Velcro straps and silicone pad may be cleaned with a liquid enzymatic detergent, such as Cabco Zyne. Please use your standard cleaning procedure for the triangles, pads, and straps.

Revised 11/24/15

FREE TRIAL ON MOST INSTRUMENTS

© 2015 Innomed, Inc.

INNOMED 



103 Estus Drive, Savannah, GA 31404
www.innomed.net info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

1.800.548.2362