

STERILIZATION PROCEDURE

Nicholson Universal Humeral Prosthesis Extractor with Case

Product No's: 3670 w/3925 Slaphammer
3670-10 Adapter Foot
3760-CABLE
3670-CASE

Please use the following guidelines when sterilizing this product:

Sterilizer Type:	Pre-vacuum Steam Sterilizer
Minimum Temperature:	132 degrees C / 270 degrees F
Minimum Cycle Time:	4 minutes
Minimum Dry Time:	30 minutes

Can be pre-vacuum steam sterilized **wrapped or unwrapped.**

Revised 11/14/17

FREE TRIAL ON MOST INSTRUMENTS

© 2017 Innomed, Inc.

INNOMED 



103 Estus Drive, Savannah, GA 31404
www.innomed.net info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

1.800.548.2362