

STERILIZATION PROCEDURE

Robb Leg Positioner

Product No: 2630

Made of aluminum and stainless steel.

Please use the following guidelines when sterilizing this product:

Sterilizer Type:	Pre-vacuum Steam Sterilizer
Minimum Temperature:	132 degrees C / 270 degrees F
Minimum Cycle Time:	4 minutes
Minimum Dry Time:	30 minutes

Can be pre-vacuum steam sterilized **wrapped or unwrapped**, with the footpiece detached from the base.

Revised 1/10/17

FREE TRIAL ON MOST INSTRUMENTS

© 2013 Innomed, Inc.

INNOMED 



103 Estus Drive, Savannah, GA 31404
www.innomed.net info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

1.800.548.2362