

## STERILIZATION PROCEDURE

# Stainless Steel Instruments

*Please use the following guidelines when sterilizing this product:*

<b>Sterilizer Type:</b>	<b>Pre-vacuum Steam Sterilizer</b>
Minimum Temperature:	132 degrees C / 270 degrees F
<b>Minimum Cycle Time:</b>	<b>4 minutes</b>
Minimum Dry Time:	10 minutes

<b>Sterilizer Type:</b>	<b>Flash Sterilizer</b>
Minimum Temperature:	132 degrees C / 270 degrees F
<b>Minimum Cycle Time:</b>	<b>4 minutes</b>
Minimum Dry Time:	0 minutes

Revised 1/1/12

FREE TRIAL ON MOST INSTRUMENTS

© 2012 Innomed, Inc.

**INNOMED** 



103 Estus Drive, Savannah, GA 31404  
www.innomed.net info@innomed.net

912.236.0000 Phone  
912.236.7766 Fax

**1.800.548.2362**