

DeOrio Calcaneal Z-Osteotomy Guide

Designed by James K. DeOrio, MD

Designed to help guide a z-osteotomy of the calcaneus


The guide frame can be attached to the calcaneus with 1.6 mm pins, and one horizontal cut made at 0° and second cut at either 5°, 10° or 15°—using the cutting block guide — to meet 3 cm horizontally from the horizontal cut to create an osteotomy wedge for removal. Designed for use with a 50.0 mm X 9.0 mm X 0.61 mm saw.



PRODUCT NO:

1377

Dimensions: 2" x 1" x .313" (5,1 x 2,5 x 0.8 cm)

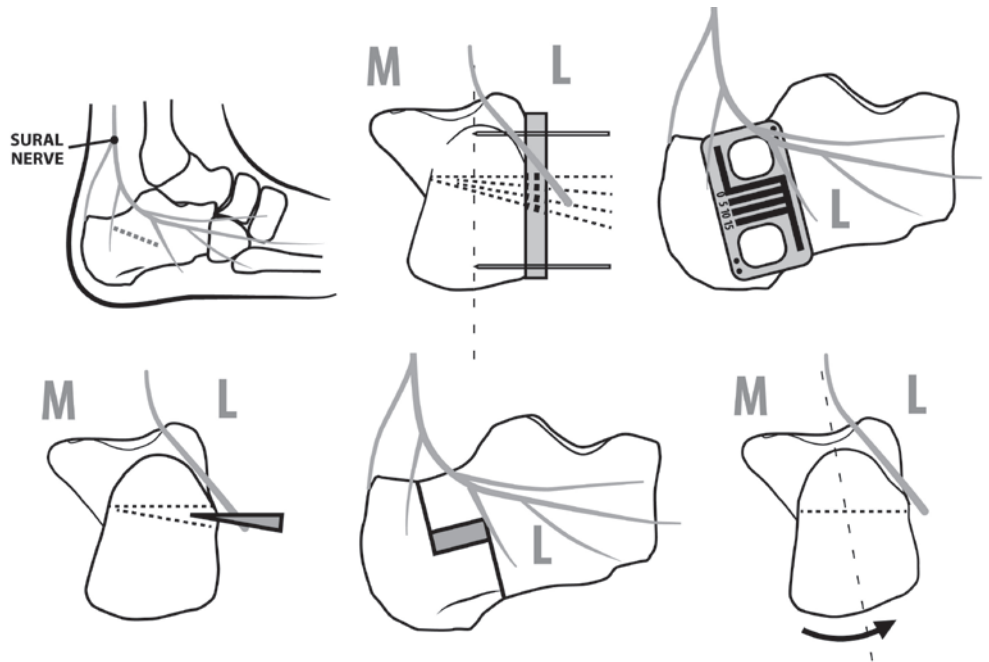

www.innomed.net



ISO 13485:2016

FREE TRIAL ON MOST INSTRUMENTS

INNOMED



103 Estus Drive, Savannah, GA 31404
info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

Innomed-Europe Tel. +41 41 740 67 74
info@innomed-europe.com

1.800.548.2362