


Femoral Head Removal Pin

Used to help remove a femoral head during total hip surgery

Partial threaded pin can be used to help remove a femoral head during total hip surgery. The pin is especially helpful in minimally invasive total hip surgery where access to the femoral head is limited. The pin is attached to a pin driver which clamps onto a Jacob chuck. When the pin is drilled in place, the driver is easily removed from the pin, as the pin is held by a friction ring. The head can be removed by gripping the pin by hand or by using a large pin inserter/extractor. Pin diameter is 5/32" (4 mm).



PRODUCT NO'S:
1310 [Pin] Overall Length: 9" (22,9 cm) Diameter: 5/32" (4 mm)
1205 [Pin Driver]
3030 [Pin Inserter/Extractor]

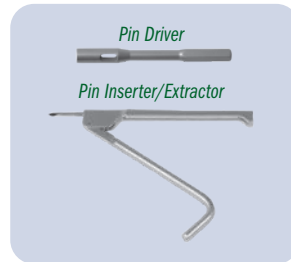

www.innomed.net

ISO 13485:2016

FREE TRIAL ON MOST INSTRUMENTS

INNOMED 



103 Estus Drive, Savannah, GA 31404
info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

Innomed-Europe Tel. +41 41 740 67 74
info@innomed-europe.com

1.800.548.2362