



Hagan Carpal Tunnel Release Sleeve


Designed by Hugh Hagan, MD

Designed to protect the surrounding anatomy while providing a sleeve within which to smoothly advance a beaver-style blade to divide and release the transverse carpal ligament

Designed to use a Beaver-style Mini-Meniscus (Flat) 4 mm Blade. Blade not included.



Designed for use in a mini-open, non-endoscopic approach, the sleeve isolates the blade, providing protection to the surrounding anatomy. The longer, bottom leading edge of the sleeve is inserted between the median nerve and the transverse carpal ligament, while the shorter, top leading edge provides lifting protection to the structures above the ligament. The blade is then advanced within the sleeve to complete the ligament release.

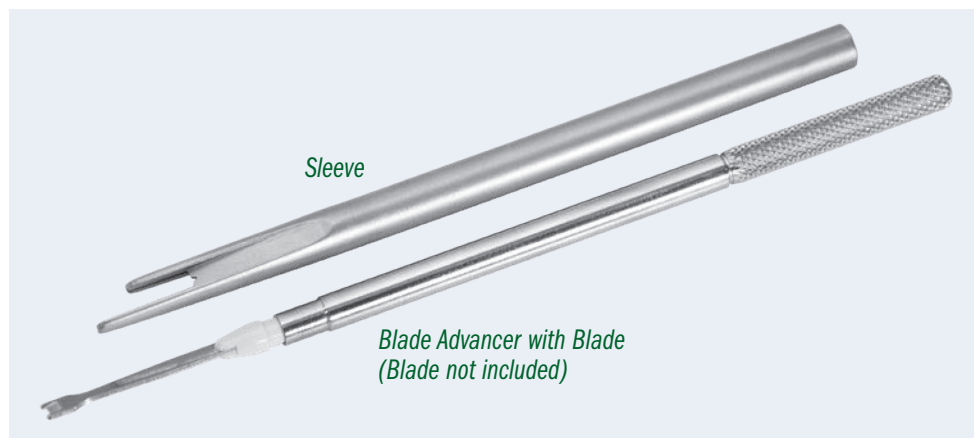

Sleeve

*Blade Advancer with Blade
(Blade not included)*

Designed to use a Beaver-style Mini-Meniscus (Flat) 4 mm Blade.
Blade not included.

PRODUCT NO:

1150

Overall Length: 5" (12,7 cm)



ISO 13485:2016

FREE TRIAL ON MOST INSTRUMENTS

INNOMED



103 Estus Drive, Savannah, GA 31404
info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

Innomed-Europe Tel. +41 41 740 67 74
info@innomed-europe.com

1.800.548.2362

© 2023 Innomed, Inc.