


# Sinha Retractor for Acetabular Reaming

Design modification by Ajoy K. Sinha, MD

*Designed to retract and protect the femur while preparing the acetabulum for reaming during antero-lateral approach total hip surgery*

After the femur is prepared and the broach has been placed, the Sinha retractor is placed on the infero-lateral aspect of the acetabulum with the neck of the broach projecting through the large hole in the retractor blade. This serves to displace the femur posteriorly and to help protect the greater trochanter while acetabular reaming is conducted.

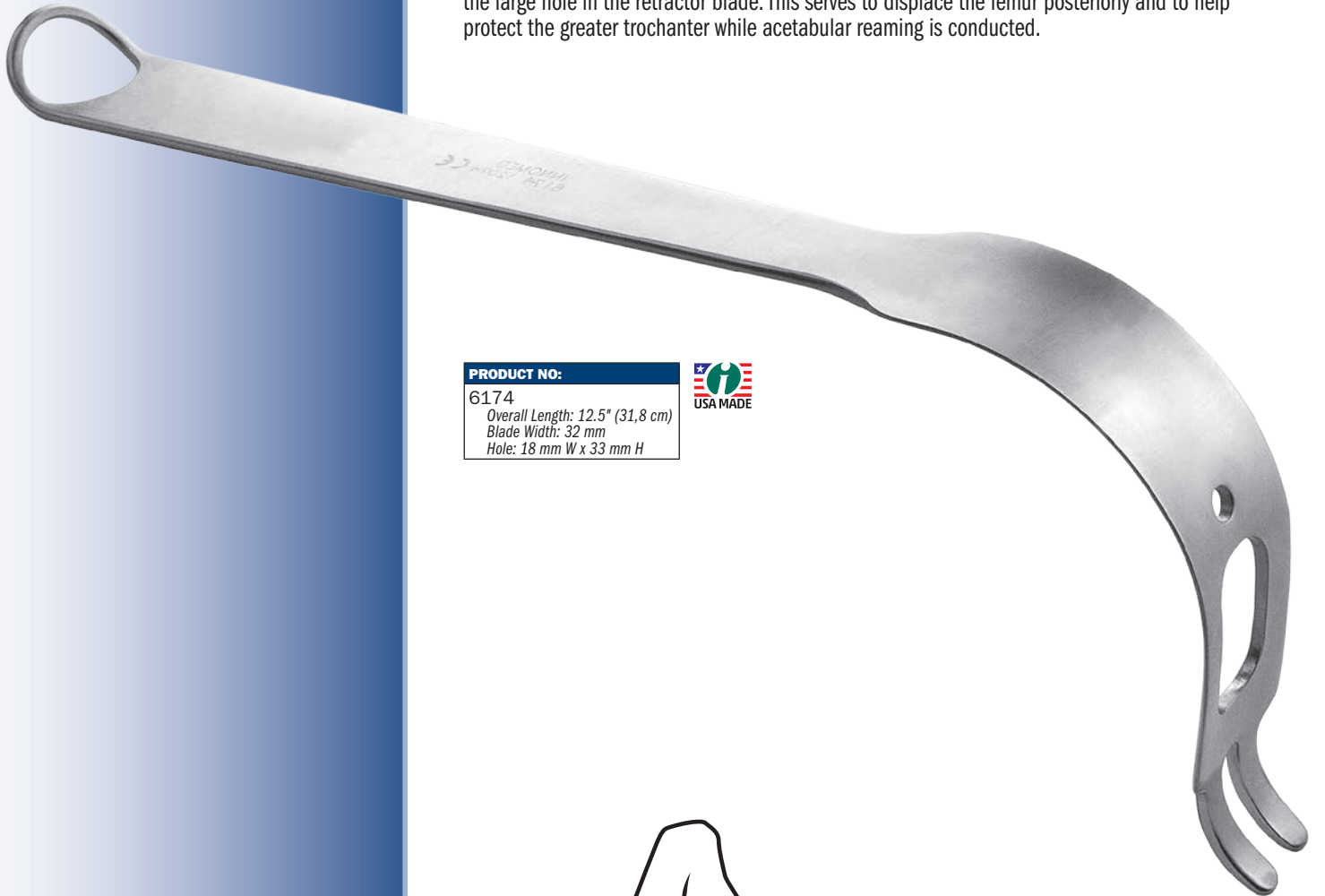


**PRODUCT NO:**

6174

Overall Length: 12.5" (31,8 cm)

Blade Width: 32 mm

Hole: 18 mm W x 33 mm H

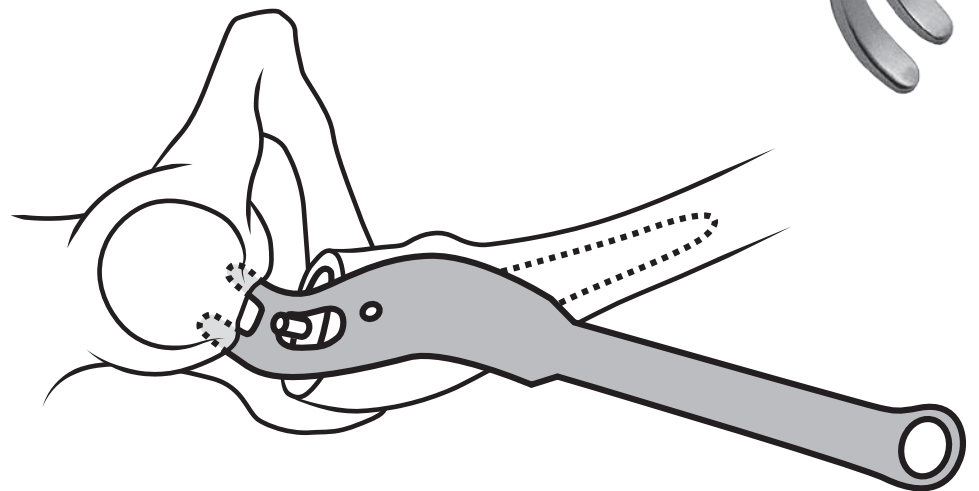

ISO 9001:2008 • ISO 13485:2003

**FREE TRIAL ON MOST INSTRUMENTS**

Scan to  
Launch Our  
Website



103 Estus Drive, Savannah, GA 31404  
www.innomed.net info@innomed.net

912.236.0000 Phone  
912.236.7766 Fax

Innomed-Europe Tel. +41 41 740 67 74  
Fax +41 41 740 67 71

**INNOMED**

**1.800.548.2362**